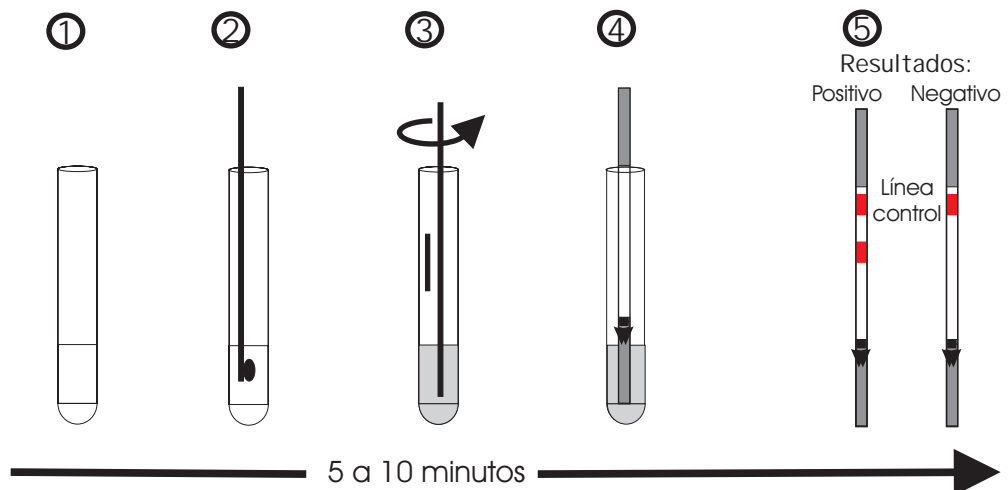
Adeno-Strip®

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Set-up of a rapid immunochromatographic diagnostic test for Adenovirus detection :

Th.Leclipteux, D. Col, M. Venuti, F. Paulart, D. Van Beers, M. De Foor and R. Viehof.

European Society for Clinical Virology. Progress in Clinical Virology IV, Hamburg, Germany, August 1998

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5 INFLAMMATION AND EXPLOSION

- 5,1 Flash not applicable
- 5,2 Auto-inhibition temperature not applicable
- 5,3 Unusual fire and explosion in case of fire, the product emits toxic fumes hazards
- 5,4 Extinguishing agents -suitable : usual means
-not suitable: none
- 5,5 Special fire fighting procedures: set a self contained respiratory system in the working area
- 5,6 Other recommendations: avoid rejection of extinguishing material in the environment

6 TOXICOLOGICAL DATA

not applicable

7 EMERGENCY FIRST AID PROCEDURES

- only with liquids contact with skin : Rinse with water
- only with liquids contact with eyes : Rinse with plenty of water for at least 20 minutes
- only with liquids inhalation : Ask medical advice
- only with liquids ingestion : Ask medical advice

8 PROTECTION OF THE ENVIRONMENT

avoid any discharge in the vicinity of fish pounds or grazing ground.

9 OTHER INFORMATION

This safety data sheet completes the package insert but cannot replace it.
Coris BioConcept is not responsible on potential hazards when the product and its components are not employed according to the recommendations of the insert.

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4 STORAGE AND MANIPULATION

4,1 Precautions to be taken in handling and storing	handling : storing :	none not above 50 °C nor below 0 °C
4,2 Packaging material	recommended:	Box: cardboard dilution bottle : glass Stick vial : PVC
	to be avoided :	metals
4,3 Dangerous reactions with:	avoid to put in contact with fire	
4,4 Hazardous decomposition product:	none	
4,5 Personal protective equipment:	gloves	
4,6 Special protection measures:	avoid ingestion, inhalation or skin contact with dilution buffer	
4,7 Measures to be taken after leaking accidental spillage:	collect samples in plastic bag and submit to sterilization or procedures. rinse carefully polluted bench	
4,7.1 Neutralization or destruction of product	-sterilization of samples and used sticks - set in specialized recuperator service	
4.7.2 Destruction of emptied packaging	-set in specialized and recuperator service	
4,8 Other recommendations	none	

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2 CONSTITUTION OF THE PRODUCT

2,1 Components	Laminate	active matrix: plastic backing :	nitrocellulose + mylar PVC
	Absorbent paper	sample pad absorbant pad	glass fiber cellulose paper
	Release matrix	conjugate pad	Polyester fiber
	Dilution buffer	SALTS : ADDITIVE(S) : CONSERVATIVE : pH : MOLARITY :	restricted information restricted information sodium azide 7,2 restricted information

2,2 Biologicals	Specific reagent	anti-Adenovirus	mouse monoclonal antibody
	Control reagent	anti-mouse IgG	Protein G purified rabbit antiserum Protein A purified
	Conjugate	anti-Adenovirus (hexon antigen)	mouse monoclonal antibody Protein G purified Gold colloï d coupled
	conservative(s)	Sodium azide	

2,3 Hazardous ingredients

None

2,4 Impurities (potentially harmful)

None